TEST PROCEDURE

- Wash hands with soap and warm water, rinse with clean water and allow to dry. If this is not possible, use the provided gauze as an alternative.
- Prepare the necessary material as follows: open the aluminium pouch, take out only the test cassette and throw away the desiccant bag. Open the plastic packet containing the pipette. – **FIG. A**
- Carefully rotate the protective cap of the sterile lancet 360° without pulling it. Extract and discard the released cap. – **FIG. B**
- Carefully massage the finger chosen for the puncture (the side of the ring finger is recommended). It is important that the massage is done from the palm of the hand to the phalanx, to improve blood flow. Press the open end of the lancet (the side the cap has been extracted from), against the fingertip – **FIG. C**. The tip of the lancet automatically retracts after use. If the lancet does not work properly, discard it and use the second one supplied. If the second one is not required, it can be disposed of without special precautions.
- Holding the hand down, massage the finger until a large drop of blood forms. It is important to massage from the palm of the hand to the phalanx to improve blood flow. – **FIG. D**
- Take the pipette **without pressing the bulb**. Two sampling methods are suggested:
 - FIG. E1:** hold the pipette horizontally **without pressing the bulb** place it in contact with the drop of blood, it will enter the pipette by capillary. Move the pipette away when the black line is reached. If there is not enough blood, continue to massage the finger until the black line is reached.
 - FIG. E2:** place the pipette on a clean, flat surface with the tip protruding from the shelf, then place the drop of blood in contact with the pipette, it will enter by capillarity. If blood is not sufficient, continue massaging the finger until the blood has reached the black line. Avoid, as far as possible, to continuously move the tip of the pipette away from the finger in order to prevent the formation of air bubbles.
- Place the blood collected with the pipette into the well indicated on the cassette (S) by pressing the pipette bulb. – **FIG. F**
- Unscrew the blue cap from the dropper vial (leave the white cap tightly screwed on). Deposit 2 drops into the well indicated on the cassette (S), wait for 5 seconds between the first and second drop. – **FIG. G**
- Wait 5 minutes and read the results referring to the next section for result interpretation.

RESULTS INTERPRETATION

READ THE RESULT AFTER EXACTLY 5 MINUTES

The intensity of the line colours is not relevant for the purposes of interpretation of the Test's results.

NEGATIVE RESULT

Two coloured bands appear in the reading window by the T (test) and C (control) signs. The T band may be less intense (lighter) than the C line. This result means the levels of ferritin in the blood are above 30 ng/ml. In some cases, samples with ferritin levels just below this value (between 27 ng/ml and 29 ng/ml) can yield a result of this type.

POSITIVE RESULTS

A coloured band appears only under the C (control) sign. This means the value of ferritin is below normal levels and medical advice is needed. Consult a physician.

NON VALID RESULT

No bands appear or there is a line only under the T (test) sign and not under the C (control) sign. In this case it is not possible to interpret the result of the test, which must be considered non valid. Repeat the test with a new blood sample.

F.A.Q. – QUESTIONS AND ANSWERS

HOW DOES THE IRON FER TEST WORK?

Ferritin is a protein responsible for storing iron in cells. A positive result means that the concentration of ferritin is ≤ 30 ng/ml. The Test was calibrated on the 3rd International Standard NIBSC 94/572. In some cases, samples with ferritin concentrations between 27 ng/ml and 29 ng/ml can yield a negative result.

WHEN CAN THE TEST BE USED? The Iron Fer Test can be performed in the presence of symptoms such as paleness, tiredness, frequent migraines and widespread pain, palpitations and occasionally increased heart rate, lack of muscle strength. The Test can be carried out at any time of the day, but must not be performed in the case of illness, acute inflammation, liver or spleen problems. In these cases, there may be positive results even in the absence of an actual iron deficiency.

WHAT SHOULD I DO IF THE RESULT IS POSITIVE? If the result is positive, the concentration of ferritin in the blood is below normal levels and you should consult a physician, who will establish what to do next.

WHAT SHOULD I DO IF THE RESULT IS NEGATIVE? If the result is negative, the concentration of ferritin is at least 30 ng/ml. In some cases, samples with ferritin levels just below this value (between 27 ng/ml and 29 ng/ml) can yield a result of this type.

CAN THE RESULT BE INCORRECT? The result is correct if the instructions are followed carefully. However, the result may not be correct if: the device comes into contact with other liquids before being used, if part of the diluent is accidentally split, if the amount of blood is not enough or the number of drops dispensed in the well is more than 3. The plastic pipette supplied allows users to be sure they have collected the right amount of blood.

HOW ACCURATE IS THE IRON FER TEST? The Test is very accurate. Assessment reports show a concordance rate of 96.6% (CI 95%: 93.4-99.8%) with reference methods. Despite the reliability of the test, falsely positive or falsely negative results are possible.

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IS THE RESULT RELIABLE IF READ AFTER 10 MINUTES? No. The test must be read 5 minutes after the procedure is completed.

WHAT SHOULD I DO IF THE RESULT IS POSITIVE? If the result is positive, the concentration of ferritin in the blood is below normal levels and you should consult a physician, who will establish what to do next.

WHAT SHOULD I DO IF THE RESULT IS NEGATIVE? If the result is negative, the concentration of ferritin is at least 30 ng/ml. In some cases, samples with ferritin levels just below this value (between 27 ng/ml and 29 ng/ml) can yield a result of this type.

CAN THE RESULT BE INCORRECT? The result is correct if the instructions are followed carefully. However, the result may not be correct if: the device comes into contact with other liquids before being used, if part of the diluent is accidentally split, if the amount of blood is not enough or the number of drops dispensed in the well is more than 3. The plastic pipette supplied allows users to be sure they have collected the right amount of blood.

HOW ACCURATE IS THE IRON FER TEST? The Test is very accurate. Assessment reports show a concordance rate of 96.6% (CI 95%: 93.4-99.8%) with reference methods. Despite the reliability of the test, falsely positive or falsely negative results are possible.

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