TEST PROCEDURE

- 1) Wash hands with soap and warm water, rinse with clean water and allow to dry. Note: The use of warm water facilitates capillary blood collection as it induces vasodilation.
2) Lay out the material needed as follows: open the aluminium bag, take out only the test strip case and dispose of the desiccant packet. Open the plastic packet containing the pipette. Open the vial by unscrewing the white cap, making sure it does not fall to the ground.
3) Carefully rotate the protective cap of the sterile lancet 360° without pulling it, then extract and discard the cap.
4) Carefully massage the finger chosen for the puncture (the side of the ring finger is recommended). It is important that the massage is done from the palm of the hand to the fingertip, to improve blood flow. Press the open end of the lancet (the side the cap has been extracted from), against the fingertip.
5) Holding the hand downwards, massage the finger until a large drop of blood forms. It is important to massage from the palm of the hand to the fingertip to improve blood flow.
6) Take the pipette without pressing the bulb. Two sampling methods are suggested.
7) Deposit the blood collected with the pipette into the previously opened vial. Press the pipette bulb 2 or 3 times to ensure that all the blood has moved into the diluent solution.
8) Unscrew the blue cap from the dropper vial (leave the white cap tightly screwed on).
9) Wait 10 minutes and then read the results as indicated.

RESULTS INTERPRETATION

READ THE RESULT AFTER EXACTLY 10 MINUTES The intensity of the line colour does not have any importance for the interpretation of the test result.

NEGATIVE RESULT

A coloured line appears only under the C (control) sign. This result means there are no anti-DGP antibodies in the blood or that their concentration is below the test's detection limit.

POSITIVE RESULTS

Two coloured bands appear in the reading window by the T (test) and C (control) signs. The intensity of the T band may be lower than that of the C band. The test must therefore be considered positive regardless of the intensity of the T test band. This result shows the presence of anti-DGP antibodies in the blood and therefore probable celiac disease.

NON VALID RESULT

No bands appear or there is a line only under the T (test) sign and not under the C (control) sign. In this case it is not possible to interpret the result of the test, which must be considered not valid. Repeat the test.

FAQ. - QUESTIONS AND ANSWERS

HOW DOES THE CELIAC TEST WORK? Gliadin is a protein component in gluten responsible for the immune reaction that causes intestinal inflammation in patients suffering from celiac disease. Recent studies of the diagnosis of celiac disease have shown that deamidated gliadin peptides in particular can trigger strong immune reactions.
WHEN CAN THE TEST BE USED? The Celiac Test can be performed when symptoms such as diarrhoea, abdominal pain and swelling, weight loss, fatigue and/or skin reactions are present.
CAN THE RESULT BE INCORRECT? The result is correct if the instructions are followed carefully.

IF I AM ON A GLUTEN-FREE DIET, CAN I TAKE THE TEST? No. In celiac disease patients on a gluten-free diet the symptoms and the level of disease-associated antibodies decrease until they are no longer detectable.
HOW DO I INTERPRET THE TEST IF THE COLOUR AND INTENSITY OF THE TEST AND CONTROL BANDS ARE DIFFERENT? The colour and the intensity of the bands are not important for the interpretation of the result.
IS THE RESULT RELIABLE IF READ AFTER 15 MINUTES? No. The test must be read 10 minutes after the procedure is completed.
WHAT SHOULD I DO IF THE RESULT IS POSITIVE? A positive result means that the levels of class IgA and class IgG anti-DGP antibodies is approximately 20 U/ml.
WHAT SHOULD I DO IF THE RESULT IS NEGATIVE? A negative result means that the levels of class IgA and class IgG anti-DGP antibodies are below the test's detection limit.

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